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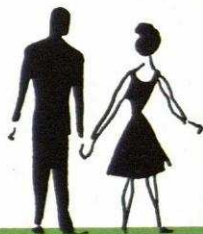
la nouvelle

méthode

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**OUI** *ou* **NON**

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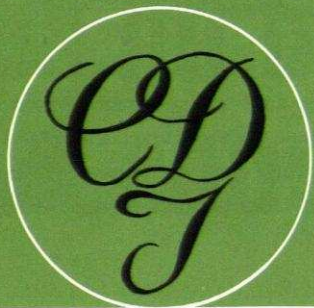
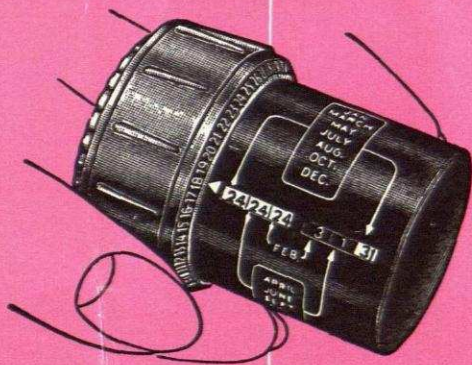
## Conception Days Indicator

### Features

- Accurately predicts the fertile period in each woman's menstrual cycle based on the Ogino-Knaus Theory.
- Can be simply and precisely set for the individual woman with shortest cycle not less than 21 days and with longest cycle not more than 38 days, and when the maximum cycle-difference is not more than 12 days.
- Is simply rotated to date when menstruation begins and the fertile period can be read off directly.
- Built-in security devices exclude mistakes in setting.

**T**HE Rhythm Method developed by Dr. Kyasaku Ogino and Professor Hermann Knaus has established that the fertile period in a woman's menstrual cycle is limited to a few days in each month. These days are related to the day when menstruation begins and the length of time between one period and the next; this is what is meant by the term „cycle“.

The cycle is measured from the first day of one period to the day before the following period, both days included (e.g. if the first day of the period is January 1st and the next period starts on January 29th, the cycle is 28 days).

To set your C. D. Indicator correctly, you should know your shortest and longest cycles over a year (e.g. if the interval between two periods was 26 days on five occasions, 27 days on six, and 29 days once, your shortest cycle is 26 and your longest 29 over a year). If you have not kept a record which gives you the exact figures, you can work it out like this: you will probably remember approximately your shortest and longest cycle—the actual month when these occur is not important. To avoid the risk of a mistake when setting your Indicator, **deduct** some days from your supposed shortest cycle and **add** some days to your supposed longest cycle as **extra** security days. Although the Indicator automatically adds security days for you, this correction—of one to three days, depending on the reliability of your memory—should be made to ensure that you have not in the last year had longer or shorter cycles than those set on your Indicator (e.g., if you remember your shortest cycle as 26 days and your longest 29 days, you should set your Indicator at 24 shortest and 31 longest). If you are using the Rhythm Method for the first time, we would recommend that you make a similar adjustment for extra security.

It is most important to keep accurate cycle records, and to help you with this we have included a calendar. With this calendar before you, you can reset the Indicator, after a time, to limit the fertile days to the period scientifically determined by Professor Knaus.

When you wish conception to occur, the intercourse has to take place in the middle of the fertile period indicated. The probability of conception is increased by the man's abstaining for 10 days before. Good health condition of the parents at this time has positive influence on the child's physical and mental qualities.

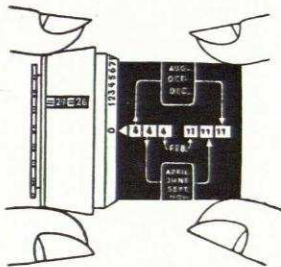
The course of your cycles could be disturbed by grave illness, nervous shock, change in climate, etc. In such a case do not use the device until your cycles show a normal range. If your cycles are disturbed for many months, consult your doctor. Check your cycles after childbirth and reset the device if necessary.

# How to use your Indicator

**1**

Hold the device by the white collar with your left hand and pull off the grey cover. Rotate the black cylindrical dial with your right hand till the triangle at the slot is opposite 0.


Now with your left hand push the collar as far as it will go to the right so as to expose the two wheels marked "Minimum" and "Maximum" both of which are numbered from 21 to 38 (Fig. 2).

**2**

Hold the cycle wheels with your left hand and rotate the collar with your right (you will notice that the black cylindrical dial will also turn) till its opening is opposite the figure of your shortest cycle (e. g. in Fig. 2 shortest cycle is 24 days). Then slide the collar – with the opening over this figure – back to cover the first toothed wheel (Fig. 3).

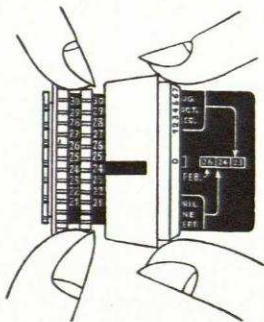Note: Automatic stop prevents setting when difference between shortest and longest cycle is more than 12 days. It also prevents the shortest and longest cycles from being confused.


**3**

Rotate collar again till its opening is opposite the figure of your longest cycle (e. g. in Fig. 3 longest cycle is 27 days). Then slide collar – with the opening over this figure – right back to its original position.

At correct setting the figures of your longest and shortest cycles are visible through the opening.

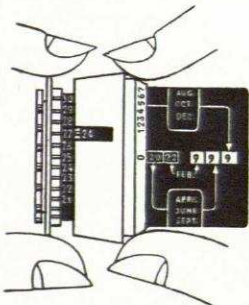The device is now adapted for your cycles and cannot open of its own accord when shifted from the 0 position.

C.D.

**INDICADOR****CALENDAR**

Record regularly the first day of your menstruation and count the days on an ordinary calendar beginning with the first day of the menstruation till and including the last day before commencement of the next menstruation. The number of days so counted represents the cycle and should be marked in the space provided for it. Then mark the shortest and the longest cycle—which you need for setting your C. D. Indicator.

For example of how to keep the record see below.

C. D. INDICATOR LTD, ZURICH

**EXAMPLE**

Beginning of Menstruation Dates	Cycles Days
1 cycle	
March 18	<b>27</b>
April 14	
2 cycle	
April 14	<b>24 x</b>
May 8	
3 cycle	
May 8	<b>25</b>
June 2	
4 cycle	
June 2	<b>26</b>
June 28	
5 cycle	
June 28	<b>29 x</b>
July 27	
6 cycle	
July 27	<b>25</b>
Aug. 21	
Shortest cycle:	<b>24</b>
Longest cycle:	<b>29</b>

Beginning of Menstruation Dates	Cycles Days
1 cycle	
2 cycle	
3 cycle	
4 cycle	
5 cycle	
6 cycle	
7 cycle	
8 cycle	
9 cycle	
10 cycle	
11 cycle	
12 cycle	
13 cycle	
14 cycle	

Beginning  
of Menstruation  
Dates

Cycles  
Days

Beginning  
of Menstruation  
Dates

Cycles  
Days

15 cycle

29 cycle

16 cycle

30 cycle

17 cycle

31 cycle

18 cycle

32 cycle

19 cycle

33 cycle

20 cycle

34 cycle

21 cycle

35 cycle

22 cycle

36 cycle

23 cycle

37 cycle

24 cycle

38 cycle

25 cycle

39 cycle

26 cycle

40 cycle

27 cycle

41 cycle

28 cycle

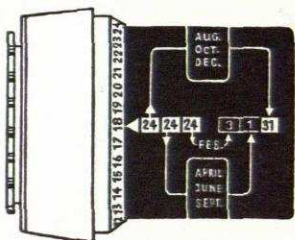
42 cycle

## Determination of the fertile period

Each month you simply rotate the black cylindrical dial so that the triangle at the slot is opposite the date when your menstruation starts. These dates, numbered from 1 to 31, are on the right-hand side of the collar.

4

The dates when conception is possible are now visible in the slots of the dial. Note the different groups of months with 31, 30 and 28 days. The three numbers in the left slot show the starting dates of the fertile period and the three numbers in the right slot the last date of the fertile period in the group of months to which the arrows refer. You follow the arrow crossing the group which includes the month when your menstruation starts.

**Black numbers on white background indicate the date in the same month as your menstruation starts, white numbers on black background the following month.**

Examples: see diagram: (Setting: shortest cycle 24, longest cycle 27)  
a) First day of menstruation Aug. 18. Fertile days: Aug. 24 till Aug. 31  
b) First day of menstruation Sept. 18. Fertile days: Sept. 24 till Oct. 1

**Note: sterile (infertile) days are all days from the onset of your menstruation till the starting date of the fertile period and from the last date of the fertile period till the next menstruation.**

N. B. During leap year 1960, 1964, 1968, when menstruation begins in February and fertile period is indicated for March, correct reading in March simply by subtracting one day from first and last date of fertile period.

Patented in most countries

### Bibliography:

Prof. Knaus, Hermann M. D.; *The Fertile and Infertile Days of Women and Their Definite Calculation*. Herbert E. Budek Co. Inc., Hackensack N. J. 1951.

Latz, Leo J., M. D. *The Rhythm of Sterility and Fertility in women*. The Latz Foundation, Chicago 1953. Published with Ecclesiastical Approbation.

Dickinson, R. L.: *Techniques of Conception Control*. Baltimore 1950. The Williams & Wilkins Co.

Speech of Pope Pius XII to the Italian and Austrian Union, 1951.

**C. D. INDICATOR LTD. ZURICH / SWITZERLAND**

C.D.Indicator

is the Symbol  
for  
happiness  
in Marriage  
throughout  
the World

## *Why Family Planning?*

Your children deserve to come into a home that is ready to welcome them. An unwanted pregnancy is dangerous to the health and happiness of the whole family, and it is every woman's right—and duty—to safeguard that health and happiness by planning the birth of her children with care and confidence. Your family deserves to be protected against the physical and financial dangers of an unplanned pregnancy, and only you can give them this protection.

And what of the emotional dangers? Is your marriage losing its romance because of the tension and anxiety you suffer, through fear of pregnancy? You and your husband deserve to have the full joy of your sexual relations without fear or restraint, the joy that will keep your love as new and meaningful as it was when you were first married.

We are fortunate to live in a day and age when medical science, through the development of the C.D. (Conception Days) Indicator, has made this possible. Isn't it up to you to take advantage of it?

## *Many unsatisfactory methods . . .*

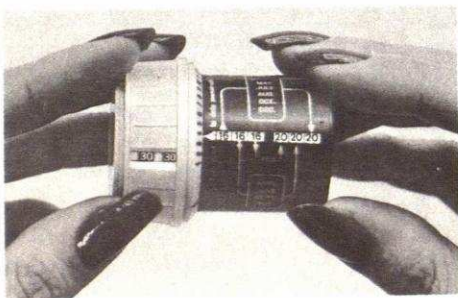
Although there are a number of contraceptives available to the woman of today, they are unsatisfactory for many reasons. Methods calling for the use of internal devices can be difficult and embarrassing; the same applies to male contraceptives. They are unnatural, and result in greatly reduced enjoyment of the love act. Many medical tests have shown them to be dangerously unreliable, and a husband and wife may suffer the untold misery of ruined sexual relations only to find that pregnancy has resulted in spite of their efforts.

On the other hand, artificial chemical contraceptives such as "The Pill" have not yet been declared completely safe, and it is not known whether they may be responsible for side effects to women and their unborn children. Furthermore, none of these unnatural methods has received the blessing of all religious denominations.

But you may be, as so many women are, only too painfully aware of these facts; now you will be happy to hear about the one answer to the family planning problem—the Conception Days Indicator.

**C.D.I. IS THE ONLY FAMILY PLANNING METHOD** that is

- (1) Completely natural
- (2) Completely safe
- (3) Simple to use
- (4) In full harmony with nature and the intimacy of the love act
- (5) Fully acceptable to all religious denominations, including the Roman Catholic church
- (6) A once-only investment.

## *What the C.D. Indicator does*

The C.D. Indicator is based on one of the most important medical discoveries of the century—the facts that women—all women—

- (1) **HAVE ONLY THREE FERTILE DAYS EVERY MONTH,** and
- (2) **CAN CONCEIVE A CHILD ONLY ON THOSE FERTILE DAYS.**

On the remaining days, completely free intercourse may be enjoyed without fear of pregnancy. That is the simple and foolproof basis of this great boon to health and happiness. By determining exactly the days on which you are fertile, the C.D. Indicator sets you free forever from the fear of an unwanted pregnancy. It is completely natural and easy to use. By simply turning a dial, you immediately see the few days of each month when you can become pregnant . . . All others are there for the full enjoyment of your marriage by you and your husband.

You deserve the joy of unspoiled marital relations. Psychologists agree that a sexually starved woman suffers in nerves, in temper, in health. A full, satisfying sex life does wonders for the morale and appearance of a woman, and improves not only her personality but her looks as well. In fact she becomes more beautiful—in complexion, in vitality, in her attraction as a woman.

You need face no more unpleasantness, no more danger to your health and marriage. The C.D. Indicator gives every woman the right to enjoy life and love in a way that was never before possible, and with the blessing of the church.

## *The Medical Facts . . .*

On another page you will read the story of the C.D.I., and we have mentioned that it is based on the discovery that women can conceive only on their fertile days. This is known throughout the medical world as the Ogino-Knaus method, and works in the following way. Please pay close attention to these facts; they will help you to understand the principle of the C.D.I. clearly.

- (1) Pregnancy can occur only when a male sperm meets a *live* female egg cell, or ovum.

- (2) A female egg cell is released from the ovary only once each month and lives just a few hours. This is known as the period of ovulation.
- (3) Therefore, the male sperm must meet the egg *during the period of ovulation* in order for pregnancy to occur. However, the fertilising life of the male sperm is only 48 hours.
- (4) Therefore, the fertile period each month for any woman is limited to the life span of the sperm—2 days, and the egg—1 day, making three days in all.

You can therefore become pregnant only during this three day period which the C.D.I. determines, in advance, every month.

## *The C.D.I. : how it came to be*

For many years the problem of a safe, natural method of birth control had been studied by scientists all over the world. Then, although their work was carried out thousands of miles apart, Dr. H. Ogino in Japan and Dr. K. Knaus in Vienna simultaneously published their findings: The basic truth about pregnancy revealed at last: that conception can only take place on the day of ovulation or on the two previous days. On all other days, intercourse may be enjoyed without fear of pregnancy. This fact had led to the further discovery that fertile days could be determined by mathematical calculations. Doctors all over the world immediately began putting the Ogino-Knaus method into practice, and very soon reports of its complete success began pouring in. Detailed statistics appear further on but here is just one example.

The biggest test ever made in birth control was carried out by the Latz Foundation in Chicago with over 50,000 married women and the theory was proved as correct.

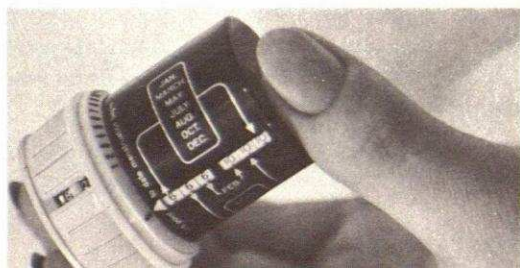
The medical profession and religious bodies greeted the news with great joy, for here at

last was a safe and acceptable planning method which would bring new happiness to every family. But there remained one major obstacle. Unfortunately, the calculations required to use the method were so complicated that only doctors and scientists could work them out, and furthermore, each woman's fertile days would have to be calculated every month on the basis of her menstrual cycle. Clearly the method would have to be made much more simple to use if it was to become available to women for their personal, individual use.

This was the problem which faced the Swiss technicians who, under the direction of a committee of famous gynaecologists, set to work designing the Conception Days Indicator. Then, after much extensive testing, the C.D.I. was perfected at last and presented to Drs. Ogino and Knaus for examination. They immediately gave the C.D.I. their full personal approval, and you can read their comments on page 12.

Now, the Ogino Knaus method could at last be put into widespread personal use. The C.D.I. was first publicly announced at the 5th Clinical Congress of Gynaecologists in America, and was greeted with such enthusiasm by the doctors present that they brought it into use in their practices almost immediately. Very soon, the simplicity and accuracy of this remarkable precision calculator proved itself to be so reliable that doctors began recommending it to patients for their personal use. Thousands of women have since relied upon the C.D.I. for its safety, its simplicity—with complete satisfaction.

That is how "C.D.I." became the symbol of health and happiness for families throughout the world.

## *What can the C.D. Indicator do for you?*

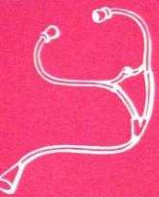
The C.D. Indicator is a Swiss-made precision calculating machine which determines by mathematics your fertile days. This remarkable little instrument follows the Ogino-Knaus method exactly, and allows automatically for any irregularity in your menstrual cycle, even for variations as great as twelve days.

In order to determine *exactly* what your fertile days will be, you simply set your own cycle information into the Indicator. That is all you need do. It will then immediately tell you your fertile days for the month. Then, once a month, just turn the marker to the date when your menstruation begins, and the fertile days for that month will immediately be shown.

Remember, the C.D. Indicator does more than simply show you your fertile days; it also takes into account any irregularity in your cycle and allows a wide margin of safety so that you can be completely sure of its accuracy.

## *And when you want a baby . . .*

By showing you your fertile days, the C.D.I. makes it possible for you to decide when your children will be born. Many husbands and wives who had previously been unable to have children have written telling us of their success with the C.D.I. method, which enabled them to prepare mentally and physically for the most opportune time of conception, and giving the best possible start for the new baby.

# The Doctors say

The following are some actual quotations of world-famed medical authorities.

**DR. K. OGINO:** *"The C.D. Indicator was shown and explained to me today. I found it a wonderful instrument which shows exactly the fertile period according to our Method."*

*Many calculators of this nature have been publicised, but to the best of my knowledge, this particular instrument has been the most ingeniously conceived of them all."*

**DR. H. KNAUS:** *"I can describe the C.D. Indicator as an inspired little masterpiece of modern technique which enables every woman to make practical use of my theory in a simple way. This apparatus is so extremely easy to handle that, after a perusal of the short description, a wrong setting appears impossible. The fertile days can be read off immediately . . . I can highly recommend this apparatus and wish it the widest distribution"*.

**DR. J. LYLE CAMERON:** *"The reason why I recommend the use of the Safe Time in preference to all other means of family limitation, where this is really necessary, is that marital relations are natural and normal in every way, and that it is free from any injurious effects"*.

**DR. H. J. GERSTER:** *"It is clear that Knaus's Theory embodies an eminently positive aspect. Every nation which values a strong and healthy rising generation should, for the well-being of its people, make this knowledge as widespread as possible"*.

**DR. L. J. LATZ:** *"Of 265 women advised by us to use the Natural Method, and who had 11,222 marital relations in 2,253 months, not one became pregnant, since intercourses were confined to their sterile or safe days"*.

**JOURNAL OF THE AMERICAN MEDICAL ASSOCIATION:** *"We have accurately kept records of 379 women who had 15,924 natural intercourses during the sterile period without the occurrence of a pregnancy"*.

# The Churches say

The Ogino-Knaus Method of Birth Control, as applied by the C.D. Indicator has the approval of all religious denominations.

His Holiness Pope Pius XII, in a letter on Christian Marriage, laid down: "*The conscious use of the 'Sterile Days' for the purpose of the cultivation of matrimonial unity and love is not against the Catholic Law of Custom*".

This position of the Vatican was made clear recently before the World Conference on Population in Geneva by Reverend Father de Lestapis.

Doctor Donald Soper, speaking for the Methodist Church of Great Britain: "*I welcome the responsible methods for controlling pregnancy as yet another, and in many ways, the greatest boon that science offers to human welfare . . .*"

George Craig Stewart, Bishop of the Episcopal Church, Chicago, states: "*I recommend and urge that all priests familiarise themselves with the Ogino/Knaus Theory so that they may be prepared to recommend it . . . And I am happy to join the Ecclesiastical Authorities of our brethren in the Roman Catholic Church in approving the method*".

The position of the Anglican Church is well known, leaving the decision to individual Christian conscience.

## People like YOU say

Thousands of letters have been received from delighted users of the C.D. Indicator and Double Control Method. Here are a few extracts—the originals, and many which we do not have space to print are, of course, available for inspection.

Mrs. W. of Loughton:

*"Two or three years back I bought one of your Indicators. I am more than pleased with it. It is everything you claim for it, and, as a practising Catholic I can say it has been the answer to all my problems"*.

Mr. P. W. of Winchester:

*"I purchased one of your C.D. Indicators and my wife and I have found it 100 per cent. reliable and have recommended it to all our friends"*.

Mrs. T. of Lynham:

*"I am a satisfied and successful possessor of your product—so much so that I require more literature about it—so that I may enlighten my friends. It is two years since I made my purchase and I have been most grateful for such intelligent help"*.

Mrs. M. of Wareham:

*"Since receiving our C.D. Indicator we have used it with great success . . ."*

Mrs. M. of Bolton:

*" . . . I am one of your grateful clients of 5 years standing and I want to pass on the method to other mothers"*.

Mrs. L. of Currock:

*" . . . I myself obtained one a year ago and found it very successful. Then I got one for a friend so this is the third time I am writing to you. Thanking you for a marvellous invention"*.

Mrs. W. of Fleet:

*"I am quite satisfied that it is of a high standard, and therefore am willing to rely on it entirely. I am confident of its success"*.

Mrs. D. of Salford:

*"It may interest you to know that my Indicator has been very useful in planning our family. We bought it seven years ago and it has never failed us yet"*.

Mrs. B. of Nottingham:

*"I am pleased to say that the Indicator is giving satisfactory service and I must say I think it is a wonderful invention"*.

Mrs. R. of Llanelly:

*"I bought my C.D. Indicator three years ago at Singapore, when my Husband was stationed there, and have used it with the utmost success ever since"*.

Mrs. C. of Birmingham:

*"I want to thank you for the C.D. Indicator you sent me before my marriage 2½ years ago. It has been very useful to me and most reliable"*.

Mrs. P. of Littlehampton:

*"Just over a year ago I purchased a C.D. Indicator from you, which I have found to be reliable and indispensable"*.

Mr. S. of Tiverton:

*"My Wife and I purchased an Indicator from you in early August. We are very favourably impressed with its simplicity, as a result we are now purchasing another for a friend"*.

Mrs. P. of Cheddar:

*"I have had an Indicator for over two years and find it marvellous in every way"*.

Mrs. M. of Belfast:

*"We purchased your Indicator when we were married and would assure you of its success. We have one little boy who is just two and I would take this opportunity of advising you of the Indicator's warm reception in our home—the best investment I have ever made"*.

Mrs. W. of Dublin:

*"I shall not hesitate to recommend the C.D.I. to my friends. My doctor already knows of it. He now recommends it to his women patients"*.

And those who wish to conceive . . .

Mrs. S. of Harpenden:

*"We have used a C.D. Indicator for some years, and in that time have had two planned babies. We have recommended the Indicator to many many people, and have never ceased to be grateful for the happiness we have gained from it"*.

Mrs. L. of Woodley:

*"My Husband and I would like to say how very satisfied we are with your C.D. Indicator. We have used it during the past ten months for controlling conception and latterly for planned conception with immediate success. We realise that it would have been impossible to work out the days of conception accurately without the aid of the Indicator. It also had a rather irregular menstrual cycle to calculate from, which adds to its great success"*.

Mrs. K. of Bridgwater:

*"We received a C.D. Indicator from you on 2/11/63 and are most happy to tell you that it has helped us to fulfil our wish for a family already"*.

Mrs. S. of York:

*"My husband and I were childless for seven years—until I discovered the C.D. Indicator. We have now a bonny boy and hope next year that he will have a little sister"*.

*A "Once-only" Investment*

*So simple . . .*

*Safe . . .*

*Convenient . . .*

## *Bibliography*

Knaus, Prof. Hermann, M.D.: *The Fertile and Infertile Days of Women and their Definite Calculation*. Herbert E. Budek Co. Inc. Hackensack, N.J. 1951.

Gerster, Doctor H. J.: *Nature's Own Birth Control*.

Lyle-Cameron, J., M.D., F.R.C.S., F.R.C.O.G.: *Motherhood and the Safe Time Simplified*.

The Lancet No. IX. Vol 2, 1956, of 1.9.56.

Speech of Pope Pius XII to the Italian and Australian Union, 1951.

*The Modern Method of Birth Control*. Grosset, Dunlop, New York.

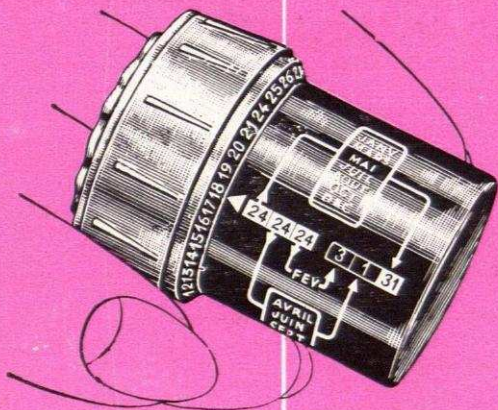
Address of Pius XII to the Italian Association Catholic Midwives, 1951.

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This additional information is being enclosed with your Indicator as we feel sure it will be of interest to you.

*With the Compliments  
of  
C. D. Indicator Ltd.*

*31, Queen Anne's Gate,  
London, S.W.1.*

## ORDER FORM

Please put a tick in the box opposite the items you require. All packages are in a plain wrapper.

- Complete Double Control method — £6 4. 6.
- C.D. Indicator only — £4 19. 6.
- C.D.I. Basal-Thermometer only — £1 7. 6.

Cheque/Postal Order/Cash for £     :     :     enclosed.

- Tick here for C.O.D. — extra charge 5/6d.

NAME.....

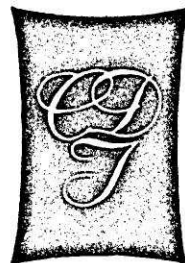
ADDRESS.....

.....

*BLOCK LETTERS PLEASE*

**C.D. INDICATOR LTD., 31, Queen Anne's Gate, S.W.1.**

T. 1076.

Personal and Confidential.

ARGENTINE  
Norca S.r.l.  
Buenos Aires

MC/VR.

AUSTRALIA  
C.D. Indicator Co.  
Sydney

AUSTRIA  
C.D. Indicator  
Vienna

BELGIUM  
Meindersma S.A.  
Brussels

BRAZIL  
Irmac Ltda.  
Rio do Janeiro

CHILE  
C.D. Indicator  
Santiago

COLOMBIA  
Putti&servicios  
Bahama

GERMANY  
C.D. Indicator  
Munich

ECUADOR  
C.D. Indicator  
Quito

FRANCE  
Laboratoires Elstar  
Paris 8e

HAITI  
Jean-Claude Chancy  
Port-au-Prince

HOLLAND  
Meindersma N.V.  
The Hague

ITALY  
Presto S.r.l.  
Milan

MALAYA  
C.D. Indicator Inc.  
Singapore

MEXICO  
C.D. Indicator  
Mexico 7, D.F.

MONACO  
Pirma S.A.  
Monte-Carlo

NEW ZEALAND  
E.S. Coult's Ltd.  
Auckland C.I.

PHILIPPINES  
C.D. Indicator  
Manila

PORTUGAL  
C.D. Indicator  
Porto

SOUTH AFRICA  
C.D. Indicator South Africa  
Johannesburg

SWITZERLAND  
C.D. Indicator AG  
Zurich

URUGUAY  
C.D. Indicator  
Montevideo

U.S.A.  
C.D. Indicator  
New York, N.Y.

VENEZUELA  
C.D. Indicator Ltda.  
Caracas

Dear Madam,

Thank you for replying to our recent advertisement. The C.D. (Conception Days) Indicator is based on one of the most important medical discoveries of our time - that conception can only take place during a few days each month.

Conception is not accidental. The C.D.I. method shows you clearly and simply the few days each month when you can conceive. This means that, for the rest of the month, you can enjoy completely natural marital relations without the restrictions and possible risks of mechanical and chemical birth control methods. These days of fertility are different for each woman because they depend on the individual dates of menstruation. However, the C.D. Indicator takes all the variations into account and builds in a vital safety factor for complete security.

The C.D.I. method is supported by world-wide medical evidence. It is recommended by doctors in more than 50 different countries and approved by all religious denominations. The enclosed leaflet gives you all the detailed information, the medical evidence and the testimonials of some of our thousands of delighted customers. After reading it, just send the order card and your Indicator will be sent, by return, in a plain package.

For hundreds of thousands of women throughout the world the arrival of the C.D. Indicator has been the happy turning point in their lives. The freedom of natural intercourse in complete safety is open to you and your husband. I do hope that you will decide to take this opportunity.

Yours sincerely,

Mary Campbell.

Personal and Confidential.

MC/VR.

Dear Madam,

Thank you for replying to our recent advertisement. The C.D. (Conception Days) Indicator is based on one of the most important medical discoveries of our time - that conception can only take place during a few days each month.

Conception is not accidental. The C.D.I. method shows you clearly and simply the few days each month when you can conceive. This means that, for the rest of the month, you can enjoy completely natural marital relations without the restrictions and possible risks of mechanical and chemical birth control methods. These days of fertility are different for each woman because they depend on the individual dates of menstruation. However, the C.D. Indicator takes all the variations into account and builds in a vital safety factor for complete security.

The C.D.I. method is supported by world-wide medical evidence. It is recommended by doctors in more than 50 different countries and approved by all religious denominations. The enclosed leaflet gives you all the detailed information, the medical evidence and the testimonials of some of our thousands of delighted customers. After reading it, just send the order card and your Indicator will be sent, by return, in a plain package.

For hundreds of thousands of women throughout the world the arrival of the C.D. Indicator has been the happy turning point in their lives. The freedom of natural intercourse in complete safety is open to you and your husband. I do hope that you will decide to take this opportunity.

Yours sincerely,

Mary Campbell.

ARGENTINE  
Norca S.r.l.  
Buenos Aires

AUSTRALIA  
C.D. Indicator Co.  
Sydney

AUSTRIA  
C.D. Indicator  
Vienna

BELGIUM  
Meindersma S.A.  
Brussels

BRAZIL  
Irmac Ltda.  
Rio do Janeiro

CHILE  
C.D. Indicator  
Santiago

COLOMBIA  
Putti&servicios  
Bahama

GERMANY  
C.D. Indicator  
Munich

ECUADOR  
C.D. Indicator  
Quito

FRANCE  
Laboratoires Elstar  
Paris 8e

HAITI  
Jean-Claude Chancy  
Port-au-Prince

HOLLAND  
Meindersma N.V.  
The Hague

ITALY  
Presto S.r.l.  
Milan

MALAYA  
C.D. Indicator Inc.  
Singapore

MEXICO  
C.D. Indicator  
Mexico 7, D.F.

MONACO  
Pirma S.A.  
Monte-Carlo

NEW ZEALAND  
E.S. Coult's Ltd.  
Auckland C.I.

PHILIPPINES  
C.D. Indicator  
Manila

PORTUGAL  
C.D. Indicator  
Porto

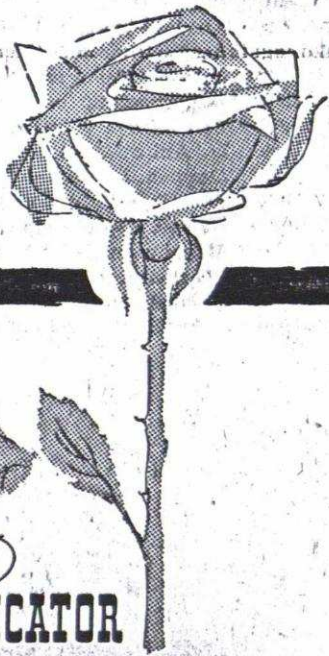
SOUTH AFRICA  
C.D. Indicator South Africa  
Johannesburg

SWITZERLAND  
C.D. Indicator AG  
Zurich

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All Rights Reserved - © by [Räber Information Management GmbH](#)