

IMPROVED CHECK PESSARY
and
CONTRACEPTIVE TABLETS
BRITISH MADE

THE "BYMESTON" METHOD—A combination of two of the most up-to-date and safest known methods.

Reasons why we claim that the "Bymeston" method is an improvement.

THE "BYMESTON" Check Pessary is an improvement upon the ordinary pessary because it has a small receptacle inside the cap, to hold a Contraceptive Tablet. When the "Bymeston" Pessary is in position, the tablet is just where it is wanted, right at the mouth of the womb. The old-fashioned method was to first adjust the cap pessary in position and then insert a soluble pessary haphazard, with the result that the soluble pessary, when melted, would run from the vagina and be lost. The "Bymeston" method remedies this serious defect, as the contraceptive tablet, being inside the cap of the pessary, cannot escape when dissolved, and furthermore, the "Bymeston" Pessary with the tablet in position, can be adjusted two or three hours beforehand if desired, without fear of the tablet being lost.

"Bymeston" Tablets are non-greasy, and of the effervescing type, forming a smooth creamy foam immediately on contact with moisture. The effect of these tablets is to fill the whole space inside the pessary with foam, which neutralizes any male sperm which may find its way past the pessary. "Bymeston" Tablets are non-irritant and a powerful antiseptic. By using them in conjunction with the "Bymeston" Improved Cap Pessary, we claim that the result should be quite certain, as, even if the pessary be carelessly adjusted, the tablet will effectively prevent conception from taking place.

DIRECTIONS FOR USE.

To fit tablet in receptacle, place one tablet on the table, and turn the pessary inside out. Now stretch the small receptacle open with the fingers, and press the pessary right down over the tablet. Should the tablet not be right inside the receptacle, it should be pressed gently. Make certain that the tablet is secure. To adjust pessary in position, hold in the right hand between thumb and finger, the two sides of the rim being pressed together. Introduce into front passage, with the hollow surface upwards. It is most important to push the rim part of the pessary as high as possible over the neck of the womb, in order to secure proper suction. The dome part of the cap is longer than the neck of the womb, and if this central part be pressed, the neck of the womb should be felt inside. If a squatting position is adopted, the Cap is easier to fix. To remove the Cap, adopt a similar position, inserting one finger under the rim, when the Cap will release (it is kept up by suction) and will easily come away. When this can be done, the handle may be cut off close to the rim, if liked, without injury to the Cap. In the case of a bride, we would mention that no Cap can be worn at all, until 2 or 3 weeks after marriage. In the meantime, the husband should wear a good Sheath, until such time as the Cap can be inserted.

The pessary, with the tablet inside, may be inserted two or three hours beforehand. Keep the pessary in position for at least 24 hours after, then take it out, and after washing well inside and out, dry carefully in a soft towel. Do not keep the pessary in position for more than two days.

These Pessaries are made in four different sizes, known as 0, 1, 2 and 3. Size 0 is suitable for newly married ladies, or those who have not given birth to a child. Sizes 1 and 2 are usually suitable for those who have already given birth to one or two children, while size 3 is usually only needed in cases where there have been three or more children. This is for the guidance of those who cannot visit a Birth Control Clinic for advice, as the success of the Pessary largely depends on the correct size being obtained. These Pessaries can be had either with or without the cord attachment.

Prices of "Bymeston" Pessaries complete with 1 dozen Tablets:—

Spring Rim	...	...	...	...	...	...	10s. 0d.
Cheaper Quality, Solid Rubber Rim	...	...	...	...	...	...	7s. 9d.
"Bymeston" Pessaries (alone) without Tablets	...	...	...	...	...	...	price 7s. 6d.
					Spring Rim		7s. 6d.
					Solid Rubber Rim		5s. 0d.
Spare Tubes of "Bymeston" Tablets	...	...	...	...	...	...	2s. 9d.

"Bymeston" Pessaries can be had either with or without the cord attachment for removal

Manufactured by the Inventors—

LAMBERTS PRORACE LIMITED,

60 & 62, QUEEN'S ROAD, DALSTON, LONDON, E.8.

Telephone: CLISSOLD 0342.

If a squatting position is adopted, the Cap is easier to fix. To remove the Cap, adopt a similar position, inserting one finger under the rim, when the Cap will release (it is kept up by suction) and will easily come away. When this can be done, the handle may be cut off close to the rim, if liked, without injury to the Cap.

In the case of a bride, we would mention that no Cap can be worn at all, until 2 or 3 weeks after marriage. In the meantime, the husband should wear a good Sheath, until such time as the Cap can be inserted.

TRADE

“BYMESTON”

MARK
REGD.

BRAND

Contraceptive and Antiseptic foaming Tablets

The only Positive and Harmless Contraceptive based on strictly Scientific Principles.

These Tablets are so manufactured that on dissolving in moisture they effervesce, and evolve hydrogen peroxide, which in contact with organic matter, is decomposed with the liberation of oxygen. It is the combined action of hydrogen peroxide and oxygen which kills the spermatozoa. It will be observed from this that a very powerful antiseptic solution is presented which has the effect of thoroughly cleansing the entire female genital tracts.

It might further be pointed out that these tablets afford protection against innumerable other risks. Also a solution of one “Bymeston” Tablet in water will give a thoroughly effective antiseptic solution which may be used periodically to cleanse the organs as occasions arise.

IMPORTANT—As “Bymeston” Tablets are not affected by climatic conditions, they are particularly suitable for use as a preventive to conception **in the Tropics**, having none of the disadvantages of the ordinary soluble quinine pessary.

DIRECTIONS FOR USE.

Whilst in a reclining position, place a “Bymeston” Tablet as high as possible into the vagina, **not less than 3 and not more than 60 minutes** before intercourse. If desired, the tablet may first be dipped in water, but this is not absolutely necessary.

Do not douche immediately after intercourse, but a douche may be used several hours afterwards if desired.

The Tablets should be kept tightly corked, and away from the light, where they will retain their properties almost indefinitely

Recommended by Numerous Doctors and Clinics.


REMEMBER—“Bymeston” Tablets relieve you of all anxiety

Price : 1 Tube 2/9, or 3 Tubes 7/6.

The "Bymeston" Tablet

The volume and thoroughness of the foam formed is adequately shown by the experiment illustrated.

1 2 3 4

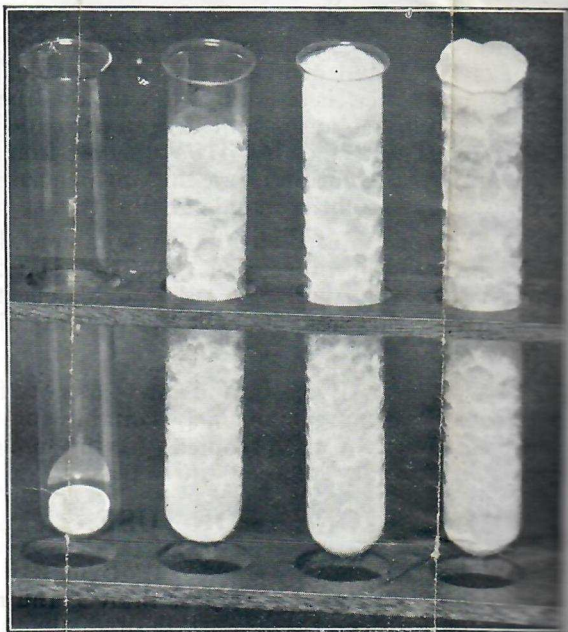

A "Bymeston" tablet was placed into each of the test tubes, and 2 c.c. of water added to tubes numbers 2, 3, and 4. The mass of foam shown was produced in space of two minutes, and the foam retained its density a considerable time. The "Bymeston" tablet in No. 1 was left intact for the purpose of comparison.

IMPORTANT—"Bymeston" tablets must be kept in a dry place, in their original tube, **tightly** corked away from the light.

BYMESTON FOAMING JELLY

A NEW PRINCIPLE
IN CONTRACEPTION

LAMBERTS PRO-RACE, LTD.
60/62, QUEEN'S ROAD
DALSTON, E.8

DESCRIBING A NEW AND BETTER CONTRACEPTIVE PREPARATION

THE BYMESTON FOAMING JELLY is an entirely new type of contraceptive, and is the result of years of research and experimental work by experts in contraceptive methods.

The preparation is an improvement on all others because it gives protection immediately the contents of the tubes are inserted. Other preparations take time to dissolve or require moisture to aid their action, and this is sometimes lacking, and these factors are the cause of many failures.

The purpose of any contraceptive method is to prevent the male sperms connecting with the female cells in the womb, and the BYMESTON FOAMING JELLY offers the simplest and safest way. Other methods exist, but none are so dependable or practicable and they have never done away with fear and worry.

Leading authorities on the subject have said that the main requirement is a highly spermicidal preparation which will not only form a barrier, but will also kill all the sperm immediately they leave the male organ. BYMESTON FOAMING JELLY is the only preparation that offers both safeguards, since other methods generally offer only one form of barrier or one type of sperm-killing substance.

BYMESTON FOAMING JELLY is the most powerful sperm-killing preparation obtainable, and yet is definitely harmless in use, and the millions of male sperms ejected each time are killed immediately they come in contact with this preparation.

It is also most important that the preparation should spread quickly and fill the numerous crevices within the female passage, so that no male sperm can lurk there and become active at the same time. BYMESTON FOAMING JELLY completely prevents this, due to

what is known as low-surface tension, certain special and new ingredients being used to achieve this penetrating effect.

This high degree of efficiency is only made possible by using the special "Tube within a Tube" which allows two preparations to be kept apart until the time for application. When the tube is pressed the contents of both tubes are squeezed out through a nozzle in which they mix and interact.

For use, the nozzle is placed within the vagina as far as possible, and the tube is then squeezed, with the result that the foam is deposited in the required position, so as to prevent pregnancy resulting from intercourse. Sufficient material for one application is contained in each tube, and in this matter medical experts have been consulted. They have stressed the importance of giving enough for all conditions, but, on the other hand, some people may find by experience that there is a little too much, and any excess may be wiped away.

The nozzle for application has likewise been improved after numerous tests, and can be used without fear of injury. This type bends easily and cannot do any harm.

In the interests of hygiene, a separate nozzle is supplied for each application, which should be thrown away afterwards. This presents a great improvement on the usual type of nozzle which is used for many applications and has to be washed each time.

Plain tubes are used to make disposal easy.

A preparation of this nature does not deteriorate even under extremes of climatic conditions.

BYMESTON FOAMING JELLY is guaranteed not to contain any irritating ingredients. It is odourless and pleasant to use.

Douching may be carried out at any time, if desired.

Price 5s. per box of six tubes and six nozzles.

Postage 4d.