

Directions for Using ELASTIC & RUBBER SHEATHS

All our Elastic and Rubber Sheaths are sold rolled up ready for use, and on no account should they be unrolled to adjust.

The proper method is to unroll the sheath on; if a sheath is too long, the surplus can be left rolled up as this will not cause inconvenience. Any attempt to draw or pull the sheath on is dangerous, as the strain will probably tear the elastic.

To remove the sheath after use, the same precaution should be taken, and it should be carefully rolled off from the open end; do not try to pull it off bodily.

Wash in cold running water without soap, then dry in a soft towel, or between the pages of clean blotting paper. When dry, dust inside and out with French chalk, and re-roll ready for use again.

Where economy is a consideration, we advise the use of our re-rolling apparatus, which may be had in three sizes; small, medium and large, the price being 2/6, post free.

LAMBERTS PRORACE LTD.

60 & 62, Queen's Road, Dalston, London, E.8.

DIRECTIONS FOR USING ELASTIC & RUBBER SHEATHS

The proper method is to roll the sheath up, then unroll it on ; if a sheath is too long, the surplus can be left rolled up as this will not cause inconvenience. Any attempt to draw or pull the sheath on is dangerous, as the strain will probably tear the elastic.

To remove the sheath after use, the same precaution should be taken, and it should be carefully rolled off from the open end ; do not try to pull it off bodily.

Wash in cold running water without soap, then dry in a soft towel, or between the pages of clean blotting paper. When dry, dust inside and out with French chalk, and re-roll ready for use again.

LAMBUTT SPECIAL SHEATH

No. 10 and 10a.

This Sheath may be used with safety 10 to 12 times.

These Sheaths may be used with safety
3 to 4 times.